

# Finishing & Polishing Products

## for Mollosil® plus and Molloplast-B®



The successful completion of high performance soft relines requires the use of the correct set of instruments and polishes, in the correct sequence, to obtain the highest quality results. The following products from Buffalo Dental, used in the suggested steps, produce the optimum final surface quality in the shortest time for Mollosil plus and Molloplast-B relines.

### **Step 1 – Reduction: SoftGrind™ Grinding Sleeves**

After removing gross amounts of excess material with scissors or a sharp blade, SoftGrind Grinding Sleeves are the ideal instruments for all-purpose trimming and finishing of the surfaces of soft reline materials. The 5/16" OD x 1/2" Long friction-fit grinding sleeves with angled tip are specially designed to grind quickly and smoothly without clogging, and may be disposed after use. The reusable rubber-tipped sleeve-holder is mounted on a convenient 3/32" diameter straight HP mandrel (with 1 1/2" gripping length) and is easy and convenient to use.



#### **Item # Description**

**62400 SoftGrind Intro Kit**, Pkg. of 10 Coarse Grinding Sleeves and 1 Reusable HP Mandrel

**62410 SoftGrind HP Mandrel Refill**, Pkg. of 1

**62420 SoftGrind Grinding Sleeves Refill**, Pkg. of 10

### **Step 2 – Finishing: Pre-Polishers**

Pre-Polishers are the ideal instruments for smoothing trimmed surfaces in preparation for final polishing. Use of the honeycomb-textured Pre-Polishers smooths the silicone surface and removes small amounts of material without gouging or heating. Wheels are 1 1/8" Diameter x 1/4" Thick.

**62200 Pre-Polisher Kit**, Pkg. of 10 Pre-Polisher Wheels and 1 Reusable HP Mandrel



### **Step 3 – Polishing: Lustrol®**

Improved **Lustrol Polish** has been reengineered for mixing in a 1:1 ratio for perfectly sealing and providing a long lasting shine on all cured Molloplast-B and Mollosil plus surfaces. The polish is easily prepared by mixing base and catalyst together and then painting onto the surfaces being protected. The polish dries quickly (5 minutes) and provides sealed, lustrous and protected surfaces.

**58600 Lustrol Polish**, Pkg. includes 6 ml. Catalyst and 6 ml. Base (1:1 mixing ratio)

**Note: Mollosil plus Intro Kits** include special Lustrol components identified as **Mollosil plus Polish**. When you run out of this specially packaged and labeled introductory Lustrol material, please reorder Lustrol Kit #58600 above for the continued perfect polishing and sealing of your Mollosil plus soft relined surfaces.



Also available for one-step trimming and gross reduction of Mollosil plus and Molloplast-B surfaces are the original **MOLLOCUTTER Steel HP Burs** in an assorted pkg. of 3 (#62280).

And for general grinding and smoothing of relined surfaces, request our original **Mollobrasive HP Stone Points** in an assorted package of 3 (#62290).



Mollocutter Burs



Mollobrasive Points

#### **Buffalo Dental Manufacturing Co., Inc.,**

129 Lafayette Dr., PO Box 678, Syosset, NY 11791-0678 USA

Tel: 516.496.7200 or 800.828.0203 Fax: 516.496.7751

[www.buffalodental.com](http://www.buffalodental.com)

562400 0706

**Please order today from  
your favorite Dealer!**